**ARCHITECT****SYSTEM****en****L6C3H9****49-2027/R1****PLEASE COMMUNICATE TO ALL LABORATORY PERSONNEL**

---

<b>Date</b>	March 2010
<b>Product</b>	ARCHITECT Anti-HCV Reagents List numbers: 6C37-20, 6C37-25 and 6C37-30 Lot numbers: All
<b>Reason</b>	<p>We have determined that S/CO values generated with ARCHITECT Anti-HCV (LN 6C37) may decrease when 25% or less of test volume is remaining, leading to:</p> <ul style="list-style-type: none"><li>• Abbott Positive Control values shift down and/or out of range low,</li><li>• Non-Abbott Positive Control values shift down and/or out of range low, and/or</li><li>• A decrease in patient result values.</li></ul> <p>Our internal studies have shown a maximum downward shift of 40% for the positive control. Current data indicates that the drop in S/CO values is due to an inhomogeneity within the assay diluent. Investigation into the cause is still ongoing and appropriate corrective actions will be implemented as soon as possible. We will update you at the conclusion of the investigation.</p>
<b>Patient Impact</b>	Patient samples may show a decrease in S/CO values comparable to the positive control. False negative results for low reactive patient samples are estimated to occur at a calculated frequency of 0.0023% in a typical blood donor population.
<b>Action</b>	<ul style="list-style-type: none"><li>• To protect the integrity of your patient results, the ARCHITECT Anti-HCV assay diluent (LN 6C37J, green bottle label) reagent component should be mixed prior to testing each day of use. Please do the following:<ul style="list-style-type: none"><li>• Wearing clean gloves, remove the septum.</li><li>• Close the bottle with a replacement cap and gently invert 30 times.</li><li>• Wearing clean gloves, carefully snap a new septum onto the top of the bottle.</li></ul></li><li>• Abbott recommends discussing the content of this letter with your laboratory management and communicating this information to the health care providers you serve, as appropriate.</li><li>• Retain this letter for your laboratory records. If you have forwarded any ARCHITECT Anti-HCV kits to other laboratories, please inform them of this Product Correction and provide a copy of this letter to them.</li></ul>
<b>Contacts</b>	We sincerely regret any inconvenience you may have experienced. If you or any of the health care providers you serve have any questions regarding this information, please contact your local area Customer Support. Refer to control number 49-2027/R1.

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