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Aim

Comparison of different antigen rapid tests using uniform sample material

Material

Pools from nasopharyngeal and oropharyngeal swabs which are used as test specimens included in an evaluation panel.

Dry swabs were directly eluted in PBS; moist swabs were already included in the transport media of various compositions. Pools are random mixtures obtained from up to 10 samples of comparable Cq values diluted 1:10 in negative samples in PBS (Cq = Quantification cycle; also reported as CT = cycle threshold). The Cq values of a pool were determined by means of different PCR assays, and the putative number of RNA copies calculated with the aid of the INSTAND standard. In the case of the PCRs used, a Cq value of 25 corresponds to around 10⁶ RNA copies/mL. The replication of the virus in cell culture as another characteristic of the samples was determined as a possible correlate for infectiousness.

Due to the high number of tests under evaluation few specimens of the first panel were replaced (resulting in panel 1 versions 1 and 2), and a new panel (panel 2) was manufactured in analogous manner. For panel 1 versions 1 and 2, 18 or 17 samples, respectively, were characterized as Cq ≤25, 23 samples as Cq 25-30, and 9 or 10 samples, respectively, as Cq ≥30; the composition of panel 2 is: 20 samples with Cq ≤25, 20 samples with Cq 25-30, and 10 samples with Cq ≥30.

Method

The pools were aliquoted, frozen, shipped, and thawed for evaluation of the tests. For each test, 50µl of the pool (panel 1 versions 1 and 2), or 25µl (panel 2) were analysed using the components of the test provided, e.g. swabs and buffer. In case of tests designed for various sample types (e.g. nasopharyngeal, nasal, saliva, etc), the test specific swab dedicated for

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Kontakt:

E-Mail: sarscov2ivd@pei.de

nasopharyngeal samples was used. Laboratories participating in the comparative evaluation included the Robert Koch-Institut, the Paul-Ehrlich-Institut, the reference laboratory for coronaviruses (Charité), and the Institute for Microbiology of the German Army (Bundeswehr).

Summary

This comparative evaluation of a large number of SARS-CoV-2 rapid antigen tests (point of care tests; POCT) of different designs and manufacturers using the uniform sample sets allows an overview of the current state of the art regarding sensitivity.

The current state of the art was defined as corresponding to a minimal sensitivity of 75% for the pools with Cq ≤ 25 . The results do not allow any conclusions regarding specificity of the tests.

Table 1 summarizes the results of those POCTs assessed as reflecting the current state of the art (alphabetical order of manufacturers).

Table 2 summarizes the results of those POCT (alphabetical order of manufacturers) which do not fulfill the sensitivity criteria and consequently are deleted from the list of reimbursable tests (see the homepage of Federal Institute for Drugs and Medical Devices (Bundesinstitut für Arzneimittel und Medizinprodukte, BfArM)).

The first part of this evaluation has already been published (1); this publication summarizes further details of the comparative evaluation.

You should be aware that this comparative evaluation can only cover a random sample of the SARS-CoV-2 rapid antigen tests listed by the BfArM, thus eligible for reimbursement, and that few other products could not (yet) be taken into account, despite the interests on the part of the manufacturers/distributors.

Contact

Email: sarscov2ivd@pei.de

(1) Scheiblaue Heinrich, Filomena Angela, Nitsche Andreas, Puyskens Andreas, Corman Victor M, Drosten Christian, Zwirgmaier Karin, Lange Constanze, Emmerich Petra, Müller Michael, Knauer Olivia, Nübling C Micha. Comparative sensitivity evaluation for 122 CE-marked rapid diagnostic tests for SARS-CoV-2 antigen, Germany, September 2020 to April 2021. Euro Surveill. 2021;26(44):pii=2100441. <https://doi.org/10.2807/1560-7917.ES.2021.26.44.2100441>

AT-Nr.	AT-Nr. Selbsttest #	Ref-Nr. *	Hersteller	Testname	Zielantigen target antigen	Sensitivität / Sensitivity				Gesamt- Sensitivität total sensitivity	Omikron Erkennung entsprechend der Bridging Prüfung Omicron detection by bridging
						Cq ≤25	Cq 25-30	Cq ≥30			
AT-No.	AT-No. self- test #	ID-No. *	Manufacturer	Test name							
AT247/21		n.a.	Innovation Biotech (Beijing) Co.,Ltd	INV BIO Antigen Rapid Test Device (Saliva)	N	95,0%	25,0%	0,0%	48,0%	Ja	
AT199/21		n.a.	Innovita (Tangshan) Biological Technology Co.; LTD.	2019-nCoV Ag Test (Latex Chromatography Assay)	N	85,0%	5,0%	0,0%	36,0%	Ja	
AT981/21 (AT574/21)		ITP16010-TC25	InTec Products, Inc.	Rapid SARS-CoV-2 Antigen Test Colloidal Gold (Nasopharyngeal/Nasal specimen)	N	100,0%	95,0%	30,0%	84,0%	Ja	
AT849/21		n.a.	Jiangsu Biopertectus Technologies Co., Ltd.	Novel Coronavirus (SARS-CoV-2) Ag Rapid Test Kit	N	100,0%	75,0%	0,0%	70,0%	Ja	
AT418/20		n.a.	Jiangsu Diagnostics Biotechnology Co., Ltd	COVID-19 Antigen Rapid Test Cassette (Colloidal Gold)	N	100,0%	78,3%	0,0%	68,0%	Ja	
AT231/20		n.a.	Jiangsu Konsung Bio-Medical Science And Technology Co., Ltd	Konsung COVID-19 Antigen Rapid Test Kit	N	100,0%	70,0%	30,0%	74,0%	Ja	
AT244/20	AT1298/21	1031-02	Jiangsu Medomics Medical Technology Co., Ltd	SARS-CoV-2-Antigen-Testkit (LFIA)	N	94,1%	21,7%	0,0%	42,0%	Ja	
AT1082/21		R220T001B0C0	Jiangsu Mole Bioscience Co., Ltd	SARS-CoV-2 Antigen Test Cassette	N	100,0%	65,0%	0,0%	66,0%	Ja	
AT883/21		CO-05	Jiangsu Well Biotech Co., Ltd.	COVID-19 Schnelltestgerät	N	100,0%	95,0%	40,0%	86,0%	Ja	
AT770/21		n.a.	Jiangxi Province JinHuan Medical Instrument Co.,LTD.	COVID-19 antigen Schnelltest (Kolloidales Gold) / Novel Coronavirus(SARS-CoV-2) Antigen Rapid Detection Kit	N	100,0%	95,0%	60,0%	90,0%	Ja	
AT882/21		n.a.	Jiaxing Wisetest Bio-Tech Co., Ltd.	Saliva SARS-CoV-2 (2019-nCoV) Antigen Test Kit (Nanocarbon Assay)	N	85,0%	15,0%	10,0%	42,0%	Ja	
AT559/21		n.a.	Jinan Babio Biotechnology Co., Ltd.	SARS-CoV-2 Antigen Rapid Detection Kit (Colloidal Gold Method)	N	100,0%	35,0%	0,0%	54,0%	Ja	
AT236/20	AT1276/21	n.a.	Joinstar Biomedical Technology Co., Ltd (CIV care impuls Vertrieb)	COVID-19 Antigen Schnelltest (Colloidal Gold)	N	100,0%	60,9%	0,0%	64,0%	Ja	
AT527/20		COV-AG-20 / G10313	JOYSBIO (Tianjin) Biotechnology Co., Ltd.	SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold) SPUCKTEST	N	100,0%	90,0%	0,0%	76,0%	Ja	
AT692/21		G10322	JOYSBIO (Tianjin) Biotechnology Co.,Ltd.	Joybio SARS-CoV-2 Antigen rapid test kit -PRO (Colloidal Gold)	N	100,0%	35,0%	0,0%	54,0%	Ja	
AT1287/21		NCV11	Koch Biotechnology (Beijing) Co., Ltd.	COVID-19 Antigen Rapid Test Kit	N	100,0%	100,0%	40,0%	88,0%	Ja	
AT127/20	AT1284/21 / AT1155/21	B63000	Labnovation Technologies, Inc.	Labnovation SARS-CoV-2 Antigen Rapid Test Kit (Immunochromatography)	N	94,1%	17,4%	0,0%	40,0%	Ja	
AT1054/21		n.a.	Lifecosm Biotech Limited	COVID-19 Rapid Test Cassette Antigen Test Kit	N	100,0%	100,0%	100,0%	100,0%	Ja	
AT455/20		PR210101	Lumigenex (Suzhou) Co., Ltd.	PocRoc SARS-CoV-2, Antigen Schnelltest Set (Kolloidales Gold)	N	100,0%	65,2%	0,0%	64,0%	Ja	
AT151/20		71110	LumiQuick Diagnostics, Inc.	Quick Profile Covid-19 Antigen Test Card	N	100,0%	91,3%	20,0%	80,0%	Ja	